## **Almindelige regler for anvendelse.**

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## **USE ONLY AS DIRECTED.**

The Litebook® Elite™ emits an intense beam of white light. DO NOT STARE DIRECTLY AT THE LIGHT SOURCE. It is NOT harmful to glance at the light for a few seconds at a time.

We encourage use of The Litebook® Elite™ while engaged in other activities, such as reading, eating, applying makeup, or working at a computer or desk. However, take care that The Litebook® Elite™ is NOT brought into contact with water or heat-producing appliances, as they may damage The Litebook® Elite™, and may pose possible fire hazards.

If you have a history of eye disease including, but not limited to, macular degeneration, have undergone laser corrective eye surgery in the past 30 days, or if you are currently taking any medications including certain antibiotics which render you photosensitive (extremely sensitive to light), consult with your ophthalmologist or health care provider before using product.

Consult a medical professional before using this product if you have been diagnosed with Seasonal Affective Disorder, depression, mood or sleep disorders or if you are taking medication for the treatment of depression and/or mood disorders (including but not limited to: fluoxetine (Prozac),

fluvoxamine (Luvox), paroxetine (Paxil), sertraline (Zoloft), venlafaxine (Effexor), nefazodone (Serzone), Wellbutrin and Zyban). Light therapy may change the effect of these types of medications.

Occasionally mild side-effects may occur which usually resolve after a few days of use, including headaches or a 'stinging' sensation in the eyes. If these or any other adverse effects not listed do not resolve after three days, discontinue use of The Litebook® Elite™ and consult your health care provider. On very rare occasions and usually only with over-use, <1% of users may experience mania (periods of abnormally and persistently elevated, expansive or irritated mood). If this occurs, discontinue using The Litebook Elite immediately.